

免疫质控品

货号: IA3111
批号: 1942EC

包装: 12 x 5 ml
效期: 2022-12

产品描述

本品适用于临床化学体外诊断的免疫质控，用于监测免疫测定的正确度或重复度。朗道供应三种浓度水平（水平 1: IA3109；水平 2: IA3110；水平 3: IA3111）。

安全预防措施和警告

本产品仅用于体外诊断。禁止用口吸。按照实验室常规预防措施对试剂进行处理。

该质控品采用人基质血清，对所有捐献者的血清均进行了 HIV(HIV1、HIV2)抗体、肝炎 B 表面抗原(HbsAg)和肝炎 C 病毒(HCV)抗体的测试，发现均呈阴性。所采用的方法均经 FDA 认证。

然而，既然没有一种方法能够完全保证其没有传染物质，因此该质控品和所有的病人样品均应当按照能够传播疾病的样品小心处理。

保存和稳定性

开瓶后，2~8℃保存。复溶后的血清如果保存在原始容器，并且无污染，可在 2~8℃保存 7 天，并可在-20℃保存 4 周，只能冻融一次。C 肽在 2~8℃可稳定 1 天。甲状旁腺素(PTH)和甲状腺球蛋白(TG)复溶后如果 2~8℃保存，要在 4 小时内检测；若-20℃保存，则须在 2 周内检测。促肾上腺皮质激素(ACTH)如果 2~8℃保存，要在 4 小时内检测。不提供促肾上腺皮质激素(ACTH)、醛固酮和 C-肽的冷冻稳定性说明。每次使用只吸取所需用量，剩余的样品不可返回原瓶。

未开瓶，2~8℃可保存至效期末。

复溶后的血清在受到细菌污染后，会导致许多组分稳定性的下降，应该将该瓶样品抛弃，重新复溶一瓶新的血清。

使用说明

该免疫质控品为冻干粉。

1. 小心打开瓶盖，往瓶中准确加入 5 ml 蒸馏水，在 15~25℃的条件下复溶，盖上瓶盖，放置 30 分钟。轻轻翻转瓶子，确保内容物完全溶解；不要振荡，避免气泡的产生。
2. 参照各个分析仪的质控说明。
3. 冷藏未使用的材料，再次使用前，确保样品充分混合均匀。

需要自备的材料

移液管 蒸馏水

赋值

每一批质控血清都要送到参考实验室，这些实验室通过独特的统计分析方法得到一个结果进行赋值。质控范围值是平均值 $\pm$ 2 S.D.，质控结果应该落在这个区间内；然而，朗道赋值系统与客户检测系统存在的差异不可避免，所以靶值书仅供参考，建议每个实验室建立自己的参考靶值和可接受范围。

该质控品可用于醛固酮项目，但只提供水平 2 和水平 3 的靶值。

雅培 Abbott Architect 的 CA19-9 检验原理为抗原抗体反应，所使用的抗体材料是 1116-NA-19-9，有报道指出 1116-NA-19-9 的浓度很高时，Abbott Architect 系统的检验结果比其它方法的结果可能更高。

注：详细赋值信息请以原版英文说明书为准，原版说明书请在英国朗道公司官网 www.randox.com 进行下载。



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
17-羟基孕酮	nmol/l	44.8	33.6	56.0	DRG ELISA
	ng/ml	14.8	11.1	18.5	
	nmol/l	34.0	25.5	42.5	DIAsource RIA
	ng/ml	11.3	8.44	14.2	
25-OH-维生素D	nmol/l	114	85.5	143	Diasorin Liaison
	nmol/l	139	104	174	Siemens Centaur XP/XPT/Classic
	nmol/l	87.4	65.6	109	Roche Vitamin D Total
	nmol/l	98.2	73.7	123	Abbott Architect
	nmol/l	113	84.8	141	Vitros ECI
	nmol/l	86.8	65.1	109	Tosoh Series
	nmol/l	84.6	63.5	106	BioMerieux Vidas
	nmol/l	93.1	69.8	116	Beckman Dxl 600/800
	nmol/l	102	76.5	128	Fujirebio Lumipulse G Series
	nmol/l	97.2	72.9	122	Beckman Access 25 OH Vitamin D Total
	nmol/l	97.4	73.1	122	Abbott Architect (5P02)/ Alinity (8P45)
	nmol/l	108	81.0	135	Diasorin Liaison XL
	nmol/l	108	81.0	135	Roche Vitamin D Total II
	nmol/l	134	101	168	Siemens Atellica IM
促肾上腺皮质激素 (ACTH)	pmol/l	44.8	33.6	56.0	Siemens Immulite 2000/2500
	pg/ml	204	153	255	
	pmol/l	27.1	20.3	33.9	Roche Cobas E411
	pg/ml	123	92.3	154	
	pmol/l	27.2	20.4	34.0	Roche Cobas 6000/8000
	pg/ml	124	92.7	155	
	pmol/l	76.0	57.0	95.0	CIS IRMA
	pg/ml	345	259	431	
	pmol/l	40.2	30.2	50.3	Siemens Immulite 1000
	pg/ml	183	137	229	
醛固酮	pmol/l	39.0	29.3	48.8	Diasorin Liaison
	pg/ml	177	133	221	
	pmol/l	50.9	38.2	63.6	Diasorin Liaison XL
	pg/ml	231	174	288	
	pmol/l	760	570	950	DRG ELISA
	pg/ml	274	206	342	
	pmol/l	212	159	265	CIS RIA
	pg/ml	76.5	57.4	95.6	
	pmol/l	898	674	1123	Immunotech RIA
	pg/ml	324	243	405	
	pmol/l	579	434	724	Diasorin Liaison
	pg/ml	209	157	261	
	pmol/l	594	446	743	Diasorin Liaison XL
	pg/ml	214	161	267	

免疫质控品-水平3

货号: IA3111/IA3112 批号: 1942EC 规格: 12 x 5ml / 4 x 5ml 效期: 2022-12-28					
范围					
分析物	单位	靶值	低值	高值	方法学
甲胎蛋白 (AFP)	KIU/l = IU/ml	166	133	199	Abbott AFP-EIA
	ng/ml	201	161	241	
	KIU/l = IU/ml	166	133	199	Abbott Architect
	ng/ml	201	161	241	
	KIU/l = IU/ml	185	148	222	BioMerieux Vidas
	ng/ml	224	179	269	
	KIU/l = IU/ml	195	154	236	Siemens Centaur XP/XPT/Classic
	ng/ml	236	186	286	
	KIU/l = IU/ml	166	133	199	Siemens Immulite 2000/2500
	ng/ml	201	161	241	
	KIU/l = IU/ml	175	140	210	Siemens Immulite 1000
	ng/ml	212	169	255	
	KIU/l = IU/ml	161	129	193	Beckman DxI800
	ng/ml	195	156	234	
	KIU/l = IU/ml	185	148	222	Roche Elecsys
	ng/ml	224	179	269	
	KIU/l = IU/ml	196	157	235	Diasorin Liaison
	ng/ml	237	190	284	
	KIU/l = IU/ml	170	136	204	Beckman Access
	ng/ml	206	165	247	
丁胺卡那霉素	KIU/l = IU/ml	163	130	196	Tosoh Series
	ng/ml	197	157	237	
	KIU/l = IU/ml	179	143	215	Vitros ECI
	ng/ml	186	149	223	
雄烯二酮	KIU/l = IU/ml	190	152	228	Roche Cobas 6000/8000
	ng/ml	230	184	276	
	KIU/l = IU/ml	285	228	342	CIS IRMA
	ng/ml	345	276	414	
β-2-微球蛋白	KIU/l = IU/ml	191	153	229	Roche Cobas E411
	ng/ml	231	185	277	
	KIU/l = IU/ml	169	135	203	Fujirebio Lumipulse G Series
	ng/ml	204	163	245	
雄烯二酮	KIU/l = IU/ml	194	153	235	Siemens Centaur CP
	ng/ml	235	186	284	
	KIU/l = IU/ml	192	154	230	Diasorin Liaison XL
	ng/ml	232	186	278	
雄烯二酮	μmol/l	24.0	19.2	28.8	Roche Cobas 6000/8000
	μg/ml	14.1	11.2	17.0	
	μmol/l	24.1	19.3	28.9	Gravimetric
	μg/ml	14.1	11.3	16.9	
雄烯二酮	nmol/l	32.7	24.5	40.9	Siemens Immulite 2000
	ng/ml	9.38	7.03	11.7	
	nmol/l	22.5	16.9	28.1	Diasorin Liaison XL
	ng/ml	6.46	4.85	8.07	
β-2-微球蛋白	μg/ml = mg/l	5.67	4.54	6.80	Abbott Architect
	μg/ml = mg/l	6.18	4.94	7.42	BioMerieux Vidas



免疫质控品-水平3

货号：IA3111/IA3112		批号：1942EC		规格：12 x 5ml / 4 x 5ml		效期：2022-12-28	
范围							
分析物	单位	靶值	低值	高值	方法学		
β-2-微球蛋白	μg/ml = mg/l	5.60	4.48	6.72	Behring Nephelometer		
	μg/ml = mg/l	4.77	3.82	5.72	Siemens Immulite 2000/2500		
	μg/ml = mg/l	5.79	4.63	6.95	Tosoh Series		
	μg/ml = mg/l	5.59	4.47	6.71	Roche Cobas 6000/8000		
	μg/ml = mg/l	4.48	3.58	5.38	Fujirebio Lumipulse G Series		
	μg/ml = mg/l	6.43	5.14	7.72	Siemens ADVIA 1200/1650/1800/2400		
C-肽	nmol/l	3.500	2.630	4.380	Siemens Immulite 2000/2500		
	ng/ml	10.6	7.94	13.3			
	nmol/l	3.220	2.420	4.030	Roche Elecsys		
	ng/ml	9.72	7.31	12.1			
	nmol/l	3.520	2.640	4.400	Roche Cobas 6000/8000		
	ng/ml	10.6	7.97	13.2			
	nmol/l	3.410	2.560	4.260	Siemens Immulite 1000		
	ng/ml	10.3	7.73	12.9			
	nmol/l	3.910	2.930	4.890	Diasorin Liaison		
	ng/ml	11.8	8.85	14.8			
	nmol/l	3.450	2.590	4.310	Roche Cobas E411		
	ng/ml	10.4	7.82	13.0			
	nmol/l	3.210	2.410	4.010	Abbott Architect		
	ng/ml	9.69	7.28	12.1			
	nmol/l	2.740	2.060	3.420	Siemens Centaur XP/XPT/Classic		
	ng/ml	8.27	6.20	10.3			
	nmol/l	3.250	2.440	4.060	Tosoh Series		
	ng/ml	9.81	7.37	12.3			
	nmol/l	3.610	2.710	4.510	Fujirebio Lumipulse G Series		
	ng/ml	10.9	8.18	13.6			
nmol/l	3.750	2.810	4.690	Diasorin Liaison XL			
ng/ml	11.3	8.48	14.1				
nmol/l	2.710	2.030	3.390	Siemens Atellica IM			
ng/ml	8.18	6.13	10.2				
CA 15-3	U/ml	83.5	66.8	100	Abbott Architect		
	U/ml	66.4	53.1	79.7	BioMerieux Vidas		
	U/ml	75.4	52.0	98.8	Siemens Centaur XP/XPT/Classic		
	U/ml	64.5	51.6	77.4	Siemens Immulite 2000/2500		
	U/ml	84.4	67.5	101	Siemens Immulite 1000		
	U/ml	27.8	22.2	33.4	Beckman Dxl800		
	U/ml	71.7	57.4	86.0	Roche Elecsys		
	U/ml	84.5	67.6	101	Diasorin Liaison		
	U/ml	26.8	21.4	32.2	Beckman Access		
	U/ml	79.1	63.3	94.9	Tosoh Series		
	U/ml	69.8	55.8	83.8	Vitros Eci		
	U/ml	75.2	60.2	90.2	Roche Cobas 6000/8000		
	U/ml	75.2	60.2	90.2	Roche Cobas E411		
	U/ml	84.1	67.3	101	Fujirebio Lumipulse G Series		
	U/ml	74.2	51.2	97.2	Siemens Centaur CP		
	U/ml	27.8	22.2	33.4	Mindray CL-2000i		



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
CA 15-3	U/ml	77.8	62.2	93.4	Diasorin Liaison XL
CA 19-9	U/ml	161	129	193	BioMerieux Vidas
	U/ml	183	126	240	Siemens Centaur XP/XPT/Classic
	U/ml	231	185	277	Siemens Immulite 2000/2500
	U/ml	200	160	240	Beckman Dxl800
	U/ml	111	88.8	133	Roche Elecsys
	U/ml	191	153	229	Diasorin Liaison
	U/ml	238	190	286	Beckman Access
	U/ml	82.9	66.3	99.0	Tosoh Series
	U/ml	229	183	275	Vitros ECI
	U/ml	401	321	481	Abbott Architect XR/XR2 kit
	U/ml	108	86.4	130	Roche Cobas 6000/8000
	U/ml	109	87.2	131	Roche Cobas E411
	U/ml	192	154	230	Fujirebio Lumipulse G Series
	U/ml	172	119	225	Siemens Centaur CP
	U/ml	100	80.0	120	Mindray CL-2000i
	U/ml	220	176	264	Diasorin Liaison XL
CA125	U/ml	220	176	264	Abbott Architect
	U/ml	177	142	212	BioMerieux Vidas
	U/ml	190	131	249	Siemens Centaur XP/XPT/Classic OV kit
	U/ml	193	133	253	Siemens Centaur XP/XPT/Classic CA125 II kit
	U/ml	185	148	222	Siemens Immulite 2000/2500
	U/ml	208	166	250	Siemens Immulite 1000
	U/ml	157	126	188	Beckman Dxl800
	U/ml	160	128	192	Roche Elecsys
	U/ml	241	193	289	Diasorin Liaison
	U/ml	161	129	193	Beckman Access
	U/ml	280	224	336	Tosoh Series
	U/ml	171	137	205	Vitros ECI
	U/ml	162	130	194	Roche Cobas E411
	U/ml	160	128	192	Roche Cobas 6000/8000
	U/ml	160	128	192	Fujirebio Lumipulse G Series
	U/ml	198	137	259	Siemens Centaur CP CA125 II kit
	U/ml	321	257	385	Mindray CL-2000i
	U/ml	220	176	264	Diasorin Liaison XL
立痛定（卡马西平）	μmol/l	69.4	55.5	83.3	Abbott Architect
	μg/ml	16.4	13.1	19.7	
	μmol/l	70.3	43.6	97.0	Siemens Centaur XP/XPT/Classic
	μg/ml	16.6	10.3	22.9	
	μmol/l	68.2	54.6	81.8	Siemens Dimension
	μg/ml	16.1	12.9	19.3	
	μmol/l	62.5	50.0	75.0	Siemens Immulite 2000/2500
	μg/ml	14.8	11.8	17.8	
	μmol/l	62.8	50.2	75.4	Siemens Immulite 1000
	μg/ml	14.8	11.9	17.7	
	μmol/l	75.6	60.5	90.7	Roche Integra
	μg/ml	17.9	14.3	21.5	



免疫质控品-水平3

货号：IA3111/IA3112		批号：1942EC		规格：12 x 5ml / 4 x 5ml		效期：2022-12-28	
范围							
分析物	单位	靶值	低值	高值	方法学		
立痛定（卡马西平）	μmol/l	63.5	50.8	76.2	干化学法Vitros		
	μg/ml	15.0	12.0	18.0			
	μmol/l	70.5	56.4	84.6	Roche Cobas 6000/8000		
	μg/ml	16.7	13.3	20.1			
	μmol/l	69.6	55.7	83.5	Beckman DxC		
	μg/ml	16.5	13.2	19.8			
	μmol/l	73.6	62.6	84.6	Beckman AU400/600/640/2700/5400		
	μg/ml	17.4	14.8	20.0			
	μmol/l	72.2	57.8	86.6	Gravimetric		
	μg/ml	17.1	13.7	20.5			
癌胚抗原(CEA)	ng/ml = μg/l	68.1	54.5	81.7	Abbott Architect		
	ng/ml = μg/l	63.8	51.0	76.6	BioMerieux Vidas		
	ng/ml = μg/l	59.6	41.1	78.1	Siemens Centaur XP/XPT/Classic		
	ng/ml = μg/l	55.4	44.3	66.5	CIS RIA Coated tube		
	ng/ml = μg/l	77.5	62.0	93.0	Siemens Immulite 2000/2500		
	ng/ml = μg/l	85.6	68.5	103	Siemens Immulite 1000		
	ng/ml = μg/l	57.2	45.8	68.6	Beckman Dxl800		
	ng/ml = μg/l	52.2	41.8	62.6	Roche Elecsys		
	ng/ml = μg/l	55.2	44.2	66.2	Diasorin Liaison		
	ng/ml = μg/l	60.2	48.2	72.2	Beckman Access		
	ng/ml = μg/l	73.9	59.1	88.7	Tosoh Series		
	ng/ml = μg/l	62.9	50.3	75.5	Vitros ECi		
	ng/ml = μg/l	51.4	41.1	61.7	Roche Cobas 6000/8000		
	ng/ml = μg/l	52.5	42.0	63.0	Roche Cobas E411		
	ng/ml = μg/l	74.5	59.6	89.4	Fujirebio Lumipulse G Series		
	ng/ml = μg/l	60.8	48.6	73.0	Boditech Med Inc i-CHROMA		
	ng/ml = μg/l	59.6	41.1	78.1	Siemens Centaur CP		
	ng/ml = μg/l	44.9	35.9	53.9	Mindray CL-2000i		
	ng/ml = μg/l	65.4	52.3	78.5	Diasorin Liaison XL		
皮质醇	nmol/l	821	616	1026	Abbott Architect		
	μg/dl	29.6	22.2	37.0			
	nmol/l	929	697	1161	BioMerieux Vidas		
	μg/dl	33.4	25.1	41.7			
	nmol/l	1062	733	1391	Siemens Centaur XP/XPT/Classic		
	μg/dl	38.2	26.4	50.0			
	nmol/l	780	585	975	CIS RIA		
	μg/dl	28.1	21.1	35.1			
	nmol/l	907	680	1134	Siemens Immulite 2000/2500		
	μg/dl	32.7	24.5	40.9			
	nmol/l	892	669	1115	Siemens Immulite 1000		
	μg/dl	32.1	24.1	40.1			
	nmol/l	907	680	1134	Beckman Dxl800		
	μg/dl	32.7	24.5	40.9			
nmol/l	997	748	1246	Roche Elecsys			
μg/dl	35.9	26.9	44.9				

免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
皮质醇	nmol/l	940	705	1175	Diasorin Liaison
	µg/dl	33.8	25.4	42.2	
	nmol/l	969	727	1211	Beckman Access
	µg/dl	34.9	26.2	43.6	
	nmol/l	1079	809	1349	Tosoh Series
	µg/dl	38.8	29.1	48.5	
	nmol/l	899	674	1124	Vitros Eci
	µg/dl	32.4	24.3	40.5	
	nmol/l	955	716	1194	Roche Cobas 6000/8000
	µg/dl	34.4	25.8	43.0	
	nmol/l	1050	788	1313	Roche Cobas E411
	µg/dl	37.8	28.4	47.2	
	nmol/l	1094	755	1433	Siemens Centaur CP
	µg/dl	39.4	27.2	51.6	
	nmol/l	1051	788	1314	Roche Cobas 4000/e411 Gen II
	µg/dl	37.8	28.4	47.2	
去表雄酮硫酸盐 (DHEA-S)	nmol/l	972	729	1215	Roche Cobas 6000/8000 Gen II
	µg/dl	35.0	26.2	43.8	
	nmol/l	963	722	1204	Roche Elecsys Gen II
	µg/dl	34.7	26.0	43.4	
	nmol/l	1030	773	1288	Diasorin Liaison XL
	µg/dl	37.1	27.8	46.4	
	µmol/l	21.6	17.3	25.9	Abbott Architect
	µmol/l	17.3	13.8	20.8	Siemens Immulite 2000/2500
	µmol/l	18.5	14.8	22.2	Siemens Immulite 1000
	µmol/l	17.5	14.0	21.0	Beckman Dxl800
	µmol/l	26.1	20.9	31.3	Roche Elecsys
地高辛	µmol/l	17.5	14.0	21.0	Beckman Access
	µmol/l	16.5	13.2	19.8	Diasorin Liaison
	µmol/l	17.8	10.7	24.9	Siemens Centaur XP/XPT/Classic
	µmol/l	26.1	20.9	31.3	Roche Cobas E411
	µmol/l	26.2	21.0	31.4	Roche Cobas 6000/8000
	µmol/l	18.4	11.0	25.8	Siemens Centaur CP
	nmol/l	4.24	3.39	5.09	Abbott Architect
	ng/ml	3.31	2.65	3.97	
	nmol/l	5.04	4.03	6.05	BioMerieux Vidas
	ng/ml	3.94	3.15	4.73	
	nmol/l	5.23	4.08	6.38	Siemens Centaur XP/XPT/Classic
	ng/ml	4.08	3.18	4.98	
	nmol/l	5.04	4.03	6.05	Siemens Dimension
	ng/ml	3.94	3.15	4.73	
	nmol/l	5.40	4.32	6.48	Siemens Immulite 2000/2500
	ng/ml	4.22	3.37	5.07	
	nmol/l	5.96	4.77	7.15	Siemens Immulite 1000
	ng/ml	4.65	3.73	5.57	
	nmol/l	4.92	3.94	5.90	Beckman Dxl800
	ng/ml	3.84	3.08	4.60	



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
地高辛	nmol/l	5.19	4.15	6.23	Roche Integra
	ng/ml	4.05	3.24	4.86	
	nmol/l	4.92	3.94	5.90	Beckman Access
	ng/ml	3.84	3.08	4.60	
	nmol/l	4.70	3.76	5.64	干化学法Vitros
	ng/ml	3.67	2.94	4.40	
	nmol/l	4.92	3.94	5.90	Roche Cobas 6000/8000
	ng/ml	3.84	3.08	4.60	
	nmol/l	5.25	4.20	6.30	Beckman AU400/600/640/2700/5400
	ng/ml	4.10	3.28	4.92	
	nmol/l	4.98	3.98	5.98	Siemens ADVIA 1200/1650/1800/2400
	ng/ml	3.89	3.11	4.67	
	nmol/l	5.33	4.26	6.40	Roche Cobas 4000/E411
	ng/ml	4.16	3.33	4.99	
乙琥胺	nmol/l	5.12	4.10	6.14	Gravimetric
	ng/ml	4.00	3.20	4.80	
乙琥胺	μmol/l	493	394	592	Gravimetric
	μg/ml	69.9	55.9	83.9	
血清铁蛋白	ng/ml = μg/l	368	294	442	Abbott Architect
	ng/ml = μg/l	269	215	323	BioMerieux Vidas
	ng/ml = μg/l	311	249	373	Roche Tinaquant
	ng/ml = μg/l	315	246	384	Siemens Centaur XP/XPT/Classic
	ng/ml = μg/l	300	240	360	Siemens Dimension
	ng/ml = μg/l	286	229	343	Siemens Immulite 2000/2500
	ng/ml = μg/l	243	194	292	Siemens Immulite 1000
	ng/ml = μg/l	232	186	278	Beckman Dxl800
	ng/ml = μg/l	337	270	404	Roche Elecsys
	ng/ml = μg/l	342	274	410	Diasorin Liaison
	ng/ml = μg/l	335	268	402	Roche Modular E170
	ng/ml = μg/l	243	194	292	Beckman Access
	ng/ml = μg/l	228	182	274	Tosoh Series
	ng/ml = μg/l	208	166	250	Vitros Eci
	ng/ml = μg/l	277	222	332	Immunotech IRMA
	ng/ml = μg/l	333	266	400	Roche Cobas E411
	ng/ml = μg/l	266	213	319	Beckman AU400/600/640/2700/5400
	ng/ml = μg/l	199	159	239	Fujirebio Lumipulse G Series
	ng/ml = μg/l	297	232	362	Siemens Centaur CP
	ng/ml = μg/l	350	280	420	Diasorin Liaison XL
叶酸	nmol/l	13.8	10.5	17.1	Abbott Architect
	ng/ml	6.09	4.63	7.55	
	nmol/l	25.6	15.4	35.8	Siemens Centaur XP/XPT/Classic (enhanced kit)
	ng/ml	11.3	6.78	15.8	
	nmol/l	12.2	9.27	15.1	Siemens Immulite 2000
	ng/ml	5.38	4.09	6.67	
	nmol/l	19.3	14.7	23.9	Beckman Dxl800
	ng/ml	8.51	6.48	10.5	



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
叶酸	nmol/l	19.0	14.4	23.6	Beckman Access
	ng/ml	8.38	6.35	10.4	
	nmol/l	28.2	21.4	35.0	Tosoh Series
	ng/ml	12.4	9.44	15.4	
	nmol/l	11.7	8.89	14.5	Vitros ECI
	ng/ml	5.16	3.92	6.40	
	nmol/l	16.9	12.8	21.0	MP Biomedicals RIA
	ng/ml	7.45	5.64	9.26	
	nmol/l	13.0	9.88	16.1	Siemens Dimension
	ng/ml	5.73	4.36	7.10	
游离 T3	pmol/l	22.9	17.2	28.6	Abbott Architect
	ng/dl	1.49	1.12	1.86	
	pg/ml	14.9	11.2	18.6	
	pmol/l	22.3	16.7	27.9	BioMerieux Vidas
	ng/dl	1.45	1.09	1.81	
	pg/ml	14.5	10.9	18.1	
	pmol/l	21.9	13.1	30.7	Siemens Centaur XP/XPT/Classic
	ng/dl	1.42	0.852	1.99	
	pg/ml	14.3	8.58	20.0	
	pmol/l	16.2	12.2	20.3	Siemens Immulite 2000/2500
	ng/dl	1.05	0.793	1.31	
	pg/ml	10.5	7.94	13.1	
	pmol/l	15.8	11.9	19.8	Siemens Immulite 1000
	ng/dl	1.03	0.774	1.29	
	pg/ml	10.3	7.75	12.9	
	pmol/l	17.4	13.1	21.8	Beckman Dxl800
	ng/dl	1.13	0.852	1.41	
	pg/ml	11.3	8.53	14.1	
	pmol/l	29.1	21.8	36.4	Roche Elecsys
	ng/dl	1.89	1.42	2.36	
	pg/ml	18.9	14.2	23.6	
	pmol/l	35.9	26.9	44.9	Diasorin Liaison
	ng/dl	2.33	1.75	2.91	
	pg/ml	23.4	17.5	29.3	
	pmol/l	19.0	14.3	23.8	Beckman Access
	ng/dl	1.24	0.930	1.55	
	pg/ml	12.4	9.31	15.5	
	pmol/l	33.3	25.0	41.6	Tosoh Series
	ng/dl	2.16	1.63	2.69	
	pg/ml	21.7	16.3	27.1	
	pmol/l	35.0	26.3	43.8	Vitros ECI
	ng/dl	2.28	1.71	2.85	
	pg/ml	22.8	17.1	28.5	
	pmol/l	28.2	21.2	35.3	Roche Cobas E411
	ng/dl	1.83	1.38	2.28	
	pg/ml	18.4	13.8	23.0	



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
游离 T3	pmol/l	29.3	22.0	36.6	Roche Cobas 6000/8000
	ng/dl	1.90	1.43	2.37	
	pg/ml	19.1	14.3	23.9	
	pmol/l	21.5	12.9	30.1	Siemens Centaur CP
	ng/dl	1.40	0.840	1.96	
	pg/ml	14.0	8.40	19.6	
	pmol/l	21.5	16.1	26.9	Mindray CL-2000i
	ng/dl	1.40	1.05	1.75	
	pg/ml	14.0	10.5	17.5	
	pmol/l	35.9	26.9	44.9	Diasorin Liaison XL
	ng/dl	2.33	1.75	2.91	
	pg/ml	23.4	17.5	29.3	
游离 T4	pmol/l	41.3	31.0	51.6	Abbott Architect
	ng/dl	3.22	2.42	4.02	
	pg/ml	32.2	24.2	40.2	
	pmol/l	41.3	31.0	51.6	Siemens Centaur XP/XPT/Classic
	ng/dl	3.22	2.42	4.02	
	pg/ml	32.2	24.2	40.2	
	pmol/l	47.7	35.8	59.6	CIS RIA Coated tube
	ng/dl	3.72	2.79	4.65	
	pg/ml	37.2	27.9	46.5	
	pmol/l	59.7	44.8	74.6	Siemens Immulite 2000/2500
	ng/dl	4.66	3.49	5.83	
	pg/ml	46.6	34.9	58.3	
	pmol/l	60.1	45.1	75.1	Siemens Immulite 1000
	ng/dl	4.69	3.52	5.86	
	pg/ml	46.9	35.2	58.6	
	pmol/l	48.4	36.3	60.5	Beckman Dxl800
	ng/dl	3.78	2.83	4.73	
	pg/ml	37.8	28.3	47.3	
	pmol/l	60.0	45.0	75.0	Roche Elecsys
	ng/dl	4.68	3.51	5.85	
	pg/ml	46.8	35.1	58.5	
	pmol/l	81.7	61.3	102	Diasorin Liaison
	ng/dl	6.37	4.78	7.96	
	pg/ml	63.7	47.8	79.6	
	pmol/l	45.1	33.8	56.4	Beckman Access
	ng/dl	3.52	2.64	4.40	
	pg/ml	35.2	26.4	44.0	
	pmol/l	60.4	45.3	75.5	Tosoh Series
	ng/dl	4.71	3.53	5.89	
	pg/ml	47.1	35.3	58.9	
	pmol/l	89.8	67.4	112	Vitros ECI
	ng/dl	7.00	5.26	8.74	
	pg/ml	70.0	52.6	87.4	



免疫质控品-水平3

货号：IA3111/IA3112		批号：1942EC		规格：12 x 5ml / 4 x 5ml		效期：2022-12-28	
范围							
分析物	单位	靶值	低值	高值	方法学		
游离 T4	pmol/l	57.9	43.4	72.4	Roche Cobas 6000/8000		
	ng/dl	4.52	3.39	5.65			
	pg/ml	45.2	33.9	56.5			
	pmol/l	47.0	35.3	58.8	Immunotech IRMA		
	ng/dl	3.67	2.75	4.59			
	pg/ml	36.7	27.5	45.9			
	pmol/l	59.1	44.3	73.9	Roche Cobas E411		
	ng/dl	4.61	3.46	5.76			
	pg/ml	46.1	34.6	57.6			
	pmol/l	62.3	46.7	77.9	Biomerieux Vidas FT4N Kit		
	ng/dl	4.86	3.64	6.08			
	pg/ml	48.6	36.4	60.8			
	pmol/l	44.1	33.1	55.1	Siemens Centaur CP		
	ng/dl	3.44	2.58	4.30			
	pg/ml	34.4	25.8	43.0			
	pmol/l	44.1	33.1	55.1	Mindray CL-2000i		
	ng/dl	3.44	2.58	4.30			
	pg/ml	34.4	25.8	43.0			
pmol/l	89.0	66.8	111	Diasorin Liaison XL			
ng/dl	6.94	5.21	8.67				
pg/ml	69.4	52.1	86.7				
卵泡刺激素（FSH）	mU/ml	54.4	43.5	65.3	Abbott Architect		
	mU/ml	73.1	58.5	87.7	BioMerieux Vidas		
	mU/ml	77.5	60.5	94.5	Siemens Centaur XP/XPT/Classic		
	mU/ml	60.5	48.4	72.6	Siemens Immulite 2000/2500		
	mU/ml	59.0	47.2	70.8	Siemens Immulite 1000		
	mU/ml	62.7	50.2	75.2	Beckman Dxl800		
	mU/ml	56.7	45.4	68.0	Roche/BM Elecsys		
	mU/ml	75.5	60.4	90.6	Diasorin Liaison		
	mU/ml	62.3	49.8	74.8	Beckman Access		
	mU/ml	56.9	45.5	68.3	Tosoh Series		
	mU/ml	45.6	36.5	54.7	Vitros ECI		
	mU/ml	58.7	47.0	70.4	Roche Cobas E411		
	mU/ml	55.7	44.6	66.8	Roche Cobas 6000/8000		
	mU/ml	48.5	38.8	58.2	Immunotech IRMA		
	mU/ml	48.9	39.1	58.7	Fujirebio Lumipulse G Series		
	mU/ml	68.1	53.1	83.1	Siemens Centaur CP		
	mU/ml	84.4	67.5	101	Diasorin Liaison XL		
	庆大霉素	μmol/l	11.0	8.80	13.2	Roche Integra	
μg/ml		5.26	4.21	6.31			
μmol/l		11.6	9.28	13.9	Siemens Dimension Vista		
μg/ml		5.54	4.44	6.64			
μmol/l		9.55	7.64	11.5	Roche Cobas 6000/8000		
μg/ml		4.56	3.65	5.47			
μmol/l	11.7	9.36	14.0	Gravimetric			
μg/ml	5.59	4.47	6.69				

免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
生长激素 (GH)	μU/ml	43.8	35.0	52.6	CIS IRMA
	ng/ml	14.6	11.7	17.5	
	μU/ml	43.8	35.0	52.6	Siemens Immulite 2000/2500
	ng/ml	14.6	11.7	17.5	
	μU/ml	43.5	34.8	52.2	Siemens Immulite 1000
	ng/ml	14.5	11.6	17.4	
	μU/ml	36.9	29.5	44.3	Beckman Dxl800
	ng/ml	12.3	9.82	14.8	
	μU/ml	38.3	30.6	46.0	Beckman Access
	ng/ml	12.8	10.2	15.4	
	μU/ml	45.8	36.6	55.0	Tosoh Series
	ng/ml	15.3	12.2	18.4	
	μU/ml	40.4	32.3	48.5	Diasorin Liaison
	ng/ml	13.5	10.8	16.2	
	μU/ml	41.5	33.2	49.8	Roche Cobas E411
	ng/ml	13.8	11.1	16.5	
	μU/ml	39.1	31.3	46.9	Roche Cobas 6000/8000
	ng/ml	13.0	10.4	15.6	
	μU/ml	41.8	33.4	50.2	IDS iSYS
	ng/ml	13.9	11.1	16.7	
免疫球蛋白 E	μU/ml	38.5	30.8	46.2	Diasorin Liaison XL
	ng/ml	12.8	10.3	15.3	
	KIU/I = IU/ml	537	430	644	Abbott Architect
	KIU/I = IU/ml	493	394	592	Beckman Immage
	KIU/I = IU/ml	554	443	665	BioMerieux Vidas
	KIU/I = IU/ml	605	363	847	Siemens Centaur XP/XPT/Classic
	KIU/I = IU/ml	570	456	684	Siemens Immulite 2000/2500
	KIU/I = IU/ml	601	481	721	Siemens Immulite 1000
	KIU/I = IU/ml	574	459	689	Roche Elecsys
	KIU/I = IU/ml	549	439	659	Pharmacia Unicap
	KIU/I = IU/ml	554	443	665	Tosoh Series
	KIU/I = IU/ml	600	480	720	Roche Cobas E411
	KIU/I = IU/ml	611	489	733	Roche Cobas 6000/8000
I胰岛素	KIU/I = IU/ml	483	386	580	Fujirebio Lumipulse G Series
	KIU/I = IU/ml	578	347	809	Siemens Centaur CP
	mU/I	24.8	18.6	31.0	Siemens Centaur XP/XPT/Classic
	μU/ml	24.8	18.6	31.0	
	mU/I	11.6	8.70	14.5	Siemens Immulite 1000
	μU/ml	11.6	8.70	14.5	
	mU/I	11.1	8.33	13.9	Siemens Immulite 2000
	μU/ml	11.1	8.33	13.9	
	mU/I	32.9	24.7	41.1	Roche Elecsys
	μU/ml	32.9	24.7	41.1	
	mU/I	26.4	19.8	33.0	Beckman Access
	μU/ml	26.4	19.8	33.0	
	mU/I	25.9	19.4	32.4	Abbott Architect
	μU/ml	25.9	19.4	32.4	



免疫质控品-水平3

货号：IA3111/IA3112		批号：1942EC		规格：12 x 5ml / 4 x 5ml		效期：2022-12-28	
范围							
分析物	单位	靶值	低值	高值	方法学		
胰岛素	mU/l	49.7	37.3	62.1	Tosoh Series		
	μU/ml	49.7	37.3	62.1			
	mU/l	33.4	25.1	41.8	Roche Cobas E411		
	μU/ml	33.4	25.1	41.8			
	mU/l	25.1	18.8	31.4	Beckman Dxl 600/800		
	μU/ml	25.1	18.8	31.4			
	mU/l	35.5	26.6	44.4	Diasorin Liaison		
	μU/ml	35.5	26.6	44.4			
	mU/l	33.8	25.4	42.3	Roche Cobas 6000/8000		
	μU/ml	33.8	25.4	42.3			
	mU/l	30.1	22.6	37.6	Fujirebio Lumipulse G Series		
	μU/ml	30.1	22.6	37.6			
	mU/l	26.8	20.1	33.5	Siemens Centaur CP		
	μU/ml	26.8	20.1	33.5			
	mU/l	35.8	26.9	44.8	Diasorin Liaison XL		
	μU/ml	35.8	26.9	44.8			
mU/l	26.9	20.2	33.6	Siemens Atellica IM			
μU/ml	26.9	20.2	33.6				
促黄体生成素(LH)	mU/ml	40.8	32.6	49.0	Abbott Architect		
	mU/ml	50.6	40.5	60.7	BioMerieux Vidas		
	mU/ml	46.5	35.8	57.2	Siemens Centaur XP/XPT/Classic		
	mU/ml	75.7	60.6	90.8	Siemens Immulite 2000		
	mU/ml	77.5	62.0	93.0	Siemens Immulite 1000		
	mU/ml	38.2	30.6	45.8	Beckman Dxl800		
	mU/ml	53.0	42.4	63.6	Roche Elecsys		
	mU/ml	62.6	50.1	75.1	Diasorin Liaison		
	mU/ml	39.2	31.4	47.0	Beckman Access		
	mU/ml	45.6	36.5	54.7	Tosoh Series		
	mU/ml	52.8	42.2	63.4	Vitros Eci		
	mU/ml	52.7	42.2	63.2	Roche Cobas E411		
	mU/ml	52.4	41.9	62.9	Roche Cobas 6000/8000		
	mU/ml	47.0	37.6	56.4	Fujirebio Lumipulse G Series		
	mU/ml	50.5	38.9	62.1	Siemens Centaur CP		
	mU/ml	56.9	45.5	68.3	Mindray CL-2000i		
	mU/ml	64.6	51.7	77.5	Diasorin Liaison XL		
	雌二醇（E2）	pmol/l	1436	1149	1723	Abbott Architect	
pg/ml		391	313	469			
pmol/l		3146	2517	3775	BioMerieux Vidas		
pg/ml		856	685	1027			
pmol/l		2790	1730	3850	Siemens Centaur XP/XPT/Classic		
pg/ml		759	471	1047			
pmol/l		2508	2006	3010	Siemens Immulite 2000/2500		
pg/ml		682	546	818			
pmol/l		2495	1996	2994	Siemens Immulite 1000		
pg/ml		679	543	815			



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
雌二醇（E2）	pmol/l	2204	1763	2645	Beckman DxI800
	pg/ml	599	480	718	
	pmol/l	1663	1330	1996	Roche Elecsys (Estradiol II)
	pg/ml	452	362	542	
	pmol/l	2394	1915	2873	Beckman Access
	pg/ml	651	521	781	
	pmol/l	2152	1722	2582	Tosoh (E2)
	pg/ml	585	468	702	
	pmol/l	2655	2124	3186	Vitros ECI
	pg/ml	722	578	866	
	pmol/l	1685	1348	2022	Roche Cobas 6000/8000 (Estradiol II)
	pg/ml	458	367	549	
	pmol/l	1704	1363	2045	Roche Cobas 4000/E411 (Estradiol II)
	pg/ml	463	371	555	
	pmol/l	2746	1703	3789	Siemens Centaur XP/XPT/Classic Enhanced (eE2)
	pg/ml	747	463	1031	
	pmol/l	2068	1654	2482	Tosoh (iE2-hsE2)
	pg/ml	562	450	674	
	pmol/l	2894	1794	3994	Siemens Centaur CP
	pg/ml	787	488	1086	
	pmol/l	2919	1810	4028	Siemens Centaur CP Enhanced (eE2)
	pg/ml	794	492	1096	
	pmol/l	2309	1847	2771	Fujirebio Lumipulse G Series
	pg/ml	628	502	754	
扑热息痛	pmol/l	1702	1362	2042	Roche Cobas 6000/8000 (Estradiol III)
	pg/ml	463	370	556	
	pmol/l	1699	1359	2039	Roche Elecsys (Estradiol III)
	pg/ml	462	370	554	
	pmol/l	1690	1352	2028	Roche Cobas 4000/E411 (Estradiol III)
	pg/ml	460	368	552	
	pmol/l	1344	1075	1613	Diasorin Liaison XL
	pg/ml	366	292	440	
	mmol/l	0.463	0.368	0.552	Roche Integra
	mg/l	69.6	55.7	83.5	
苯巴比妥	mmol/l	0.500	0.400	0.600	Abbott Architect
	mg/l	75.7	60.5	90.9	
	mmol/l	0.499	0.400	0.600	Roche Cobas 6000/8000
	mg/l	75.7	60.5	90.9	
	mmol/l	0.529	0.420	0.630	Gravimetric
	mg/l	80.0	63.5	95.3	
	μmol/l	123	98.4	148	Abbott Architect
	μg/ml	28.5	22.8	34.2	
	μmol/l	121	96.8	145	Roche Cobas 6000/8000
	μg/ml	28.1	22.5	33.7	
	μmol/l	122	97.6	146	Siemens Immulite 2000/2500
	μg/ml	28.3	22.6	34.0	



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
苯巴比妥	μmol/l	117	93.6	140	Beckman AU400/600/640/2700/5400
	μg/ml	27.1	21.7	32.5	
	μmol/l	161	117	205	Siemens Centaur XP/XPT/Classic
	μg/ml	37.4	27.3	47.5	
	μmol/l	116	92.8	139	Roche Integra
	μg/ml	26.9	21.5	32.3	
	μmol/l	122	97.6	146	Siemens Immulite 1000
	μg/ml	28.3	22.6	34.0	
	μmol/l	113	90.4	136	Beckman DxC
	μg/ml	26.2	21.0	31.4	
	μmol/l	121	96.8	145	Gravimetric
	μg/ml	28.1	22.5	33.6	
苯妥英钠	μmol/l	93.5	74.8	112	Abbott Architect
	μg/ml	23.6	18.9	28.3	
	μmol/l	116	90.5	142	Siemens Centaur XP/XPT/Classic
	μg/ml	29.3	22.9	35.7	
	μmol/l	107	85.6	128	Siemens Dimension
	μg/ml	27.0	21.6	32.4	
	μmol/l	87.9	70.3	105	Siemens Immulite 2000/2500
	μg/ml	22.2	17.8	26.6	
	μmol/l	90.1	72.1	108	Siemens Immulite 1000
	μg/ml	22.8	18.2	27.4	
	μmol/l	92.9	74.3	111	Roche Integra
	μg/ml	23.5	18.8	28.2	
	μmol/l	96.8	77.4	116	干化学法Vitros
	μg/ml	24.4	19.5	29.3	
	μmol/l	92.7	74.2	111	Roche Cobas 6000/8000
	μg/ml	23.4	18.7	28.1	
	μmol/l	90.6	72.5	109	Beckman DxC
	μg/ml	22.9	18.3	27.5	
扑米酮	μmol/l	32.7	26.2	39.2	Gravimetric
	μg/ml	7.14	5.72	8.56	
孕酮	nmol/l	90.7	72.6	109	Abbott Architect
	ng/ml	28.4	22.7	34.1	
	nmol/l	150	120	180	BioMerieux Vidas
	ng/ml	47.0	37.6	56.4	
	nmol/l	74.5	52.9	96.1	Siemens Centaur XP/XPT/Classic
	ng/ml	23.3	16.5	30.1	
	nmol/l	69.3	55.4	83.2	Siemens Immulite 2000/2500
	ng/ml	21.7	17.3	26.1	
	nmol/l	80.2	64.2	96.2	Siemens Immulite 1000
	ng/ml	25.1	20.1	30.1	
	nmol/l	100	80.0	120	Beckman Dxl Prog Cal 33555
	ng/ml	31.3	25.0	37.6	



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
孕酮	nmol/l	103	82.4	124	Roche Elecsys (Prog. II)
	ng/ml	32.2	25.8	38.6	
	nmol/l	100	80.0	120	Beckman Access Prog Cal 33555
	ng/ml	31.3	25.0	37.6	
	nmol/l	114	91.2	137	Tosoh Series
	ng/ml	35.7	28.5	42.9	
	nmol/l	107	85.6	128	Vitros ECI
	ng/ml	33.5	26.8	40.2	
	nmol/l	103	82.4	124	Roche Cobas 4000/e411 (Prog. II)
	ng/ml	32.2	25.8	38.6	
	nmol/l	106	84.8	127	Roche Cobas 6000/8000 (Prog. II)
	ng/ml	33.2	26.5	39.9	
	nmol/l	86.6	69.3	104	Diasorin Liaison
	ng/ml	27.1	21.7	32.5	
	nmol/l	99.0	79.2	119	Beckman Dxl Prog DE Cal A80773
	ng/ml	31.0	24.8	37.2	
	nmol/l	101	80.8	121	Beckman Access Prog DE Cal A80773
	ng/ml	31.6	25.3	37.9	
	nmol/l	77.5	55.0	100	Siemens Centaur CP
	ng/ml	24.3	17.3	31.3	
	nmol/l	106	84.8	127	Mindray CL-2000i
	ng/ml	33.2	26.5	39.9	
	nmol/l	103	82.4	124	Fujirebio Lumipulse G Series
	ng/ml	32.2	25.8	38.6	
催乳素 (PRL)	nmol/l	105	84.0	126	Roche Cobas 6000/8000 (Prog. III)
	ng/ml	32.9	26.3	39.5	
	nmol/l	104	83.2	125	Roche Elecsys (Prog. III)
	ng/ml	32.6	26.0	39.2	
	nmol/l	104	83.2	125	Roche Cobas 4000/e411 (Prog. III)
	ng/ml	32.6	26.0	39.2	
	μU/ml	1108	886	1330	Abbott Architect
	μU/ml	1290	1032	1548	BioMerieux Vidas
	μU/ml	914	686	1142	Siemens Centaur XP/XPT/Classic
	μU/ml	948	758	1138	Siemens Immulite 2000/2500
	μU/ml	1003	802	1204	Siemens Immulite 1000
	μU/ml	856	685	1027	Beckman Dxl800
	μU/ml	1029	823	1235	Roche Elecsys
	μU/ml	1100	880	1320	Diasorin Liaison
	μU/ml	879	703	1055	Beckman Access
	μU/ml	1005	804	1206	Tosoh Series
	μU/ml	917	734	1100	Vitros ECI
	μU/ml	1024	819	1229	Roche Elecsys/E170/Cobas 6000/8000 PRLII
	μU/ml	1011	809	1213	Immunotech IRMA
	μU/ml	1061	849	1273	Roche Cobas E411
	μU/ml	1019	815	1223	Roche Cobas 6000/8000
	μU/ml	1243	994	1492	Fujirebio Lumipulse G Series

免疫质控品-水平3

货号: IA3111/IA3112 批号: 1942EC 规格: 12 x 5ml / 4 x 5ml 效期: 2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
催乳素	μU/ml	943	707	1179	Siemens Centaur CP
	μU/ml	1230	984	1476	Mindray CL-2000i
	μU/ml	1146	917	1375	Diasorin Liaison XL
游离PSA	ng/ml = μg/l	28.5	21.4	35.6	Abbott Architect
	ng/ml = μg/l	21.1	15.8	26.4	Siemens Centaur XP/XPT/Classic
	ng/ml = μg/l	22.4	16.8	28.0	Siemens Immulite 2000/2500
	ng/ml = μg/l	19.2	14.4	24.0	Siemens Immulite 1000
	ng/ml = μg/l	25.1	18.8	31.4	Beckman DXI standardised to Hybritech
	ng/ml = μg/l	17.1	12.8	21.4	Roche Elecsys
	ng/ml = μg/l	13.9	10.4	17.4	Diasorin Liaison
	ng/ml = μg/l	25.7	19.3	32.1	Beckman Access standardised to Hybritech
	ng/ml = μg/l	16.6	12.5	20.8	Tosoh Series
	ng/ml = μg/l	17.3	13.0	21.6	Roche Cobas E411
	ng/ml = μg/l	17.2	12.9	21.5	Roche Cobas 6000/8000
	ng/ml = μg/l	25.3	19.0	31.6	Beckman Dxl800 standardised to WHO IRP96/670
	ng/ml = μg/l	29.0	21.8	36.3	Beckman Access standardised to WHO IRP96/670
	ng/ml = μg/l	9.02	6.77	11.3	Ortho Vitros 3600/5600/ECi PSA II
	ng/ml = μg/l	18.5	13.9	23.1	Fujirebio Lumipulse G Series
	ng/ml = μg/l	12.9	9.68	16.1	Diasorin Liaison XL
PSA (总)	ng/ml = μg/l	27.2	20.4	34.0	Abbott Architect
	ng/ml = μg/l	31.7	23.8	39.6	BioMerieux Vidas
	ng/ml = μg/l	31.4	23.6	39.3	bioMerieux VIDAS TPSA
	ng/ml = μg/l	31.2	23.4	39.0	Siemens Centaur XP/XPT/Classic
	ng/ml = μg/l	13.2	9.90	16.5	Siemens Immulite 2000/2500 1st generation
	ng/ml = μg/l	12.2	9.15	15.3	Siemens Immulite 2000/2500 3rd generation
	ng/ml = μg/l	13.6	10.2	17.0	Siemens Immulite 1000 1st generation
	ng/ml = μg/l	39.3	29.5	49.1	Beckman Dxl standardisd to Hybritech
	ng/ml = μg/l	34.3	25.7	42.9	Beckman Dxl800 standardised to WHO IRP96/670
	ng/ml = μg/l	36.8	27.6	46.0	Roche Elecsys
	ng/ml = μg/l	19.2	14.4	24.0	Diasorin Liaison
	ng/ml = μg/l	40.0	30.0	50.0	Beckman Access standardised to Hybritech
	ng/ml = μg/l	40.0	30.0	50.0	Beckman Access standardised to WHO IRP96/670
	ng/ml = μg/l	23.2	17.4	29.0	Tosoh Series
	ng/ml = μg/l	36.1	27.1	45.1	Vitros ECI
	ng/ml = μg/l	34.6	26.0	43.3	CIS RIA Coated tube
	ng/ml = μg/l	36.6	27.5	45.8	Roche Cobas E411
	ng/ml = μg/l	36.4	27.3	45.5	Roche Cobas 6000/8000
	ng/ml = μg/l	25.1	18.8	31.4	Fujirebio Lumipulse G Series
	ng/ml = μg/l	36.5	27.4	45.6	Ortho Vitros 3600/5600/ECi PSA II
水杨酸	mmol/l	2.73	2.18	3.28	Roche Integra
	mg/dl	37.7	30.1	45.3	
	mmol/l	2.90	2.32	3.48	Abbott Architect
	mg/dl	40.1	32.0	48.2	
	mmol/l	2.88	2.30	3.46	Gravimetric
	mg/dl	39.8	31.8	47.8	



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
性激素结合球蛋白 (SHBG)	nmol/l	38.6	30.9	46.3	Siemens Immulite 1000
	nmol/l	44.3	35.4	53.2	Siemens Immulite 2000/2500
	nmol/l	45.6	36.5	54.7	Roche Elecsys
	nmol/l	42.6	34.1	51.1	Abbott Architect
	nmol/l	44.5	35.6	53.4	Roche Cobas E411
	nmol/l	44.5	35.6	53.4	Roche Cobas 6000/8000
	nmol/l	43.7	35.0	52.4	Beckman Access
	nmol/l	41.6	33.3	49.9	Beckman Dxl800
甲状腺素结合力	%U	38.7	31.0	46.4	Siemens Immulite 2000
睾酮	nmol/l	27.7	22.2	33.2	Abbott Architect
	ng/ml	7.98	6.39	9.57	
	ng/dl	798	639	957	
	nmol/l	22.3	17.8	26.8	BioMerieux Vidas
	ng/ml	6.42	5.13	7.71	
	ng/dl	642	513	771	
	nmol/l	23.8	17.4	30.2	Siemens Centaur XP/XPT/Classic
	ng/ml	6.85	5.00	8.70	
	ng/dl	685	500	870	
	nmol/l	32.9	26.3	39.5	Siemens Immulite 2000/2500
	ng/ml	9.48	7.57	11.4	
	ng/dl	948	757	1139	
	nmol/l	47.3	37.8	56.8	Siemens Immulite 1000
	ng/ml	13.6	10.9	16.3	
	ng/dl	1362	1089	1635	
	nmol/l	23.2	18.6	27.8	Beckman Dxl800
	ng/ml	6.68	5.36	8.00	
	ng/dl	668	536	800	
	nmol/l	25.9	20.7	31.1	Beckman Access
	ng/ml	7.46	5.96	8.96	
	ng/dl	746	596	896	
	nmol/l	40.1	32.1	48.1	Tosoh Series
	ng/ml	11.5	9.24	13.8	
	ng/dl	1155	924	1386	
	nmol/l	40.2	32.2	48.2	Vitros ECI
	ng/ml	11.6	9.27	13.9	
	ng/dl	1158	927	1389	
	nmol/l	28.6	22.9	34.3	Roche Cobas 6000 Test 2
	ng/ml	8.24	6.60	9.88	
	ng/dl	824	660	988	
	nmol/l	28.5	22.8	34.2	Roche Elecsys Test II
	ng/ml	8.21	6.57	9.85	
	ng/dl	821	657	985	
	nmol/l	28.2	22.6	33.8	Roche Cobas 4000/e411 Test II
	ng/ml	8.12	6.51	9.73	
	ng/dl	812	651	973	



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
睾酮	nmol/l	27.8	22.2	33.4	Abbott Architect 2nd Generation
	ng/ml	8.01	6.39	9.63	
	ng/dl	801	639	963	
	nmol/l	23.0	16.8	29.2	Siemens Centaur CP
	ng/ml	6.62	4.83	8.41	
	ng/dl	662	483	841	
	nmol/l	22.5	18.0	27.0	BioMerieux Vidas Testosterone II
	ng/ml	6.48	5.18	7.78	
	ng/dl	648	518	778	
	nmol/l	39.6	31.7	47.5	Mindray CL-2000i
	ng/ml	11.4	9.13	13.7	
	ng/dl	1140	913	1367	
	nmol/l	24.8	18.1	31.5	Siemens Centaur TST II
	ng/ml	7.14	5.21	9.07	
	ng/dl	714	521	907	
	nmol/l	12.0	9.60	14.4	Diasorin Liaison XL
	ng/ml	3.46	2.76	4.16	
	ng/dl	346	276	416	
茶碱	μmol/l	148	118	178	Siemens Centaur XP/XPT/Classic
	μg/ml	26.7	21.4	32.0	
	μmol/l	165	132	198	Roche Integra
	μg/ml	29.7	23.8	35.6	
	μmol/l	206	165	247	干化学法Vitros
	μg/ml	37.1	29.7	44.5	
	μmol/l	144	115	173	Abbott Architect
	μg/ml	25.9	20.7	31.1	
	μmol/l	157	126	188	Roche Cobas 6000/8000
	μg/ml	28.3	22.7	33.9	
	μmol/l	158	126	190	Beckman DxC
	μg/ml	28.5	22.7	34.3	
	μmol/l	145	116	174	Beckman AU400/600/640/2700/5400
	μg/ml	26.1	20.9	31.3	
	μmol/l	166	133	199	Gravimetric
	μg/ml	29.9	24.0	35.9	
甲状腺球蛋白（TG）	ng/ml	22.3	16.7	27.9	Siemens Immulite 1000
	ng/ml	26.5	19.9	33.1	Siemens Immulite 2000
	ng/ml	59.3	44.5	74.1	Roche Elecsys
	ng/ml	50.1	37.6	62.6	Beckman Dxl800
	ng/ml	58.1	43.6	72.6	Roche Cobas E411
	ng/ml	57.6	43.2	72.0	Roche Cobas 6000/8000
	ng/ml	52.4	39.3	65.5	Beckman Access
	ng/ml	57.8	43.4	72.3	Abbott Architect 2000i
	ng/ml	53.2	39.9	66.5	Tosoh Series
	ng/ml	57.3	43.0	71.6	Roche Cobas 6000 TGII
	ng/ml	55.3	41.5	69.1	Roche Elecsys TGII
	ng/ml	57.9	43.4	72.4	Roche Cobas 4000/E411 TGII



免疫质控品-水平3

货号：IA3111/IA3112		批号：1942EC		规格：12 x 5ml / 4 x 5ml		效期：2022-12-28	
范围							
分析物	单位	靶值	低值	高值	方法学		
甲状腺球蛋白（TG）	ng/ml	27.3	20.5	34.1	Diasorin Liaison XL		
促甲状腺激素（TSH）	μU/ml = mIU/l	17.5	14.0	21.0	Abbott Architect		
	μU/ml = mIU/l	21.5	17.2	25.8	bioMerieux VIDAS TSH		
	μU/ml = mIU/l	20.9	16.7	25.1	bioMerieux VIDAS TSH3 Ultrasensitive		
	μU/ml = mIU/l	22.5	15.3	29.7	Siemens Centaur XP/XPT/Classic		
	μU/ml = mIU/l	22.4	17.9	26.9	CIS IRMA		
	μU/ml = mIU/l	21.8	14.8	28.8	Siemens Centaur XP/XPT/Classic 3rd Generation		
	μU/ml = mIU/l	20.5	16.4	24.6	Siemens Dimension		
	μU/ml = mIU/l	21.6	17.3	25.9	Siemens Immulite 2000/2500		
	μU/ml = mIU/l	22.0	17.6	26.4	Siemens Immulite 1000		
	μU/ml = mIU/l	19.4	15.5	23.3	Beckman Dxl800 Fast TSH		
	μU/ml = mIU/l	19.0	15.2	22.8	Beckman Dxl800		
	μU/ml = mIU/l	20.3	16.2	24.4	Roche Elecsys		
	μU/ml = mIU/l	25.9	20.7	31.1	Diasorin Liaison		
	μU/ml = mIU/l	18.8	15.0	22.6	Beckman Access Fast TSH		
	μU/ml = mIU/l	19.4	15.5	23.3	Beckman Access hyperTSH 3rd Generation		
	μU/ml = mIU/l	22.5	18.0	27.0	Tosoh Series		
	μU/ml = mIU/l	28.5	22.8	34.2	Vitros ECI		
	μU/ml = mIU/l	20.6	16.5	24.7	Roche Cobas E411		
	μU/ml = mIU/l	20.7	16.6	24.8	Roche Cobas 6000/8000		
	μU/ml = mIU/l	18.7	15.0	22.4	Beckman Dxl800 Hyper TSH		
	μU/ml = mIU/l	22.6	15.4	29.8	Siemens Centaur XP/XPT/Classic TSH3-Ultra		
	μU/ml = mIU/l	21.3	14.5	28.1	Siemens Centaur CP		
	μU/ml = mIU/l	22.7	15.4	30.0	Siemens Centaur CP TSH3-Ultra		
	μU/ml = mIU/l	25.9	20.7	31.1	Mindray CL-2000i		
	μU/ml = mIU/l	19.0	15.2	22.8	Beckman Dxl 600/800 Access (3rd IS)		
	μU/ml = mIU/l	26.4	21.1	31.7	Diasorin Liaison XL		
妥布霉素	μmol/l	10.3	8.24	12.4	Gravimetric		
	μg/ml	4.82	3.86	5.80			
总 β-hCG	mU/ml=IU/l	159	127	191	Abbott Architect		
	IU/ml	0.159	0.127	0.191			
	mU/ml=IU/l	692	554	830	BioMerieux Vidas		
	IU/ml	0.692	0.554	0.830			
	mU/ml=IU/l	118	70.8	165	Siemens Centaur XP/XPT/Classic		
	IU/ml	0.118	0.071	0.165			
	mU/ml=IU/l	783	626	940	Siemens Dimension		
	IU/ml	0.783	0.626	0.940			
	mU/ml=IU/l	1076	861	1291	Siemens Immulite 2000		
	IU/ml	1.08	0.861	1.30			
	mU/ml=IU/l	950	760	1140	Siemens Immulite 1000		
	IU/ml	0.950	0.760	1.14			
	mU/ml=IU/l	146	117	175	Beckman Dxl800		
	IU/ml	0.146	0.117	0.175			
	mU/ml=IU/l	158	126	190	Diasorin Liaison		
	IU/ml	0.158	0.126	0.190			
	mU/ml=IU/l	358	286	430	Roche Cobas Core EIA		
	IU/ml	0.358	0.286	0.430			



免疫质控品-水平3

货号: IA3111/IA3112 批号: 1942EC 规格: 12 x 5ml / 4 x 5ml 效期: 2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
总 β-hCG	mU/ml=IU/l	135	108	162	Beckman Access
	IU/ml	0.135	0.108	0.162	
	mU/ml=IU/l	687	550	824	Tosoh Series
	IU/ml	0.687	0.550	0.824	
	mU/ml=IU/l	733	586	880	Vitros ECI
	IU/ml	0.733	0.586	0.880	
	mU/ml=IU/l	248	198	298	Roche STAT (Intact)
	IU/ml	0.248	0.198	0.298	
	mU/ml=IU/l	361	289	433	Roche hCG+Beta
	IU/ml	0.361	0.289	0.433	
	mU/ml=IU/l	129	103	155	Fujirebio Lumipulse G Series
	IU/ml	0.129	0.103	0.155	
	mU/ml=IU/l	141	113	169	Beckman Access Total BhCG (5th IS)
	IU/ml	0.141	0.113	0.169	
	mU/ml=IU/l	146	117	175	Beckman Dxl Total BhCG (5th IS)
	IU/ml	0.146	0.117	0.175	
总 T3	mU/ml=IU/l	118	70.8	165	Siemens Centaur CP
	IU/ml	0.118	0.071	0.165	
	mU/ml=IU/l	168	134	202	Diasorin Liaison XL
	IU/ml	0.168	0.134	0.202	
	nmol/l	5.23	3.92	6.54	Abbott Architect
	ng/ml	3.40	2.55	4.25	
	ng/dl	340	255	425	
	nmol/l	6.04	4.53	7.55	BioMerieux Vidas
	ng/ml	3.93	2.95	4.91	
	ng/dl	393	295	491	
	nmol/l	7.09	5.32	8.86	Siemens Centaur XP/XPT/Classic
	ng/ml	4.62	3.47	5.77	
	ng/dl	462	347	577	
	nmol/l	6.75	5.06	8.44	Siemens Immulite 2000/2500
	ng/ml	4.39	3.29	5.49	
	ng/dl	439	329	549	
	nmol/l	6.95	5.21	8.69	Siemens Immulite 1000
	ng/ml	4.52	3.39	5.65	
	ng/dl	452	339	565	
	nmol/l	5.94	4.46	7.43	Beckman Dxl800
	ng/ml	3.87	2.90	4.84	
	ng/dl	387	290	484	
	nmol/l	7.02	5.27	8.78	Roche Elecsys
	ng/ml	4.57	3.43	5.71	
	ng/dl	457	343	571	
	nmol/l	6.09	4.57	7.61	Beckman Access
	ng/ml	3.96	2.98	4.94	
	ng/dl	396	298	494	
	nmol/l	6.23	4.67	7.79	Tosoh Series
	ng/ml	4.06	3.04	5.08	
	ng/dl	406	304	508	



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
总 T3	nmol/l	7.20	5.40	9.00	Vitros ECI
	ng/ml	4.69	3.52	5.86	
	ng/dl	469	352	586	
	nmol/l	4.99	3.74	6.24	CIS RIA Coated tube
	ng/ml	3.25	2.43	4.07	
	ng/dl	325	243	407	
	nmol/l	6.90	5.18	8.63	Roche Cobas E411
	ng/ml	4.49	3.37	5.61	
	ng/dl	449	337	561	
	nmol/l	6.42	4.82	8.03	Roche Cobas 6000/8000
	ng/ml	4.18	3.14	5.22	
	ng/dl	418	314	522	
	nmol/l	6.07	4.55	7.59	ELISA
	ng/ml	3.95	2.96	4.94	
	ng/dl	395	296	494	
	nmol/l	7.67	5.75	9.59	Siemens Centaur CP
	ng/ml	4.99	3.74	6.24	
	ng/dl	499	374	624	
	nmol/l	4.99	3.74	6.24	Diasorin Liaison XL
	ng/ml	3.25	2.43	4.07	
	ng/dl	325	243	407	
总 T4	nmol/l	169	127	211	Abbott Architect
	µg/dl	13.2	9.91	16.5	
	ng/ml	132	99.1	165	
	nmol/l	201	151	251	BioMerieux Vidas
	µg/dl	15.7	11.8	19.6	
	ng/ml	157	118	196	
	nmol/l	189	132	246	Siemens Centaur XP/XPT/Classic
	µg/dl	14.7	10.3	19.1	
	ng/ml	147	103	191	
	nmol/l	178	134	223	Siemens Immulite 2000/2500
	µg/dl	13.9	10.5	17.3	
	ng/ml	139	105	173	
	nmol/l	179	134	224	Siemens Immulite 1000
	µg/dl	14.0	10.5	17.5	
	ng/ml	140	105	175	
	nmol/l	179	134	224	Beckman DxI800
	µg/dl	14.0	10.5	17.5	
	ng/ml	140	105	175	
	nmol/l	168	126	210	Roche Elecsys
	µg/dl	13.1	9.83	16.4	
	ng/ml	131	98.3	164	
	nmol/l	182	137	228	Beckman Access
	µg/dl	14.2	10.7	17.7	
	ng/ml	142	107	177	



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
总 T4	nmol/l	166	125	208	Tosoh Series
	µg/dl	12.9	9.75	16.1	
	ng/ml	129	97.5	161	
	nmol/l	180	135	225	Vitros ECI
	µg/dl	14.0	10.5	17.5	
	ng/ml	140	105	175	
	nmol/l	152	114	190	MP Biomedicals RIA
	µg/dl	11.9	8.89	14.9	
	ng/ml	119	88.9	149	
	nmol/l	193	145	241	CIS RIA Coated tube
	µg/dl	15.1	11.3	18.9	
	ng/ml	151	113	189	
	nmol/l	167	125	209	Roche Cobas E411
	µg/dl	13.0	9.75	16.3	
	ng/ml	130	97.5	163	
	nmol/l	167	125	209	Roche Cobas 6000/8000
	µg/dl	13.0	9.75	16.3	
	ng/ml	130	97.5	163	
	nmol/l	128	96.0	160	Immunotech IRMA
	µg/dl	9.98	7.49	12.5	
	ng/ml	100	74.9	125	
	nmol/l	177	133	221	ELISA
	µg/dl	13.8	10.4	17.2	
	ng/ml	138	104	172	
	nmol/l	251	188	314	Boditech Med Inc i-CHROMA
	µg/dl	19.6	14.7	24.5	
	ng/ml	196	147	245	
丙戊酸	nmol/l	190	133	247	Siemens Centaur CP
	µg/dl	14.8	10.4	19.2	
	ng/ml	148	104	192	
	nmol/l	137	103	171	Diasorin Liaison XL
	µg/dl	10.7	8.03	13.4	
	ng/ml	107	80.3	134	
	µmol/l	925	740	1110	Abbott Architect
	µg/ml	133	107	159	
	µmol/l	978	782	1174	Siemens Centaur XP/XPT/Classic
	µg/ml	141	113	169	
	µmol/l	913	730	1096	Siemens Dimension
	µg/ml	132	105	159	
	µmol/l	987	790	1184	Siemens Immulite 2000/2500
	µg/ml	142	114	170	
	µmol/l	1001	801	1201	Siemens Immulite 1000
	µg/ml	144	116	172	
	µmol/l	914	731	1097	Roche Integra
	µg/ml	132	105	159	



免疫质控品-水平3

货号: IA3111/IA3112 批号: 1942EC 规格: 12 x 5ml / 4 x 5ml 效期: 2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
丙戊酸	μmol/l	997	798	1196	Roche Cobas E411
	μg/ml	144	115	173	
	μmol/l	948	758	1138	Roche Cobas 6000/8000
	μg/ml	137	109	165	
	μmol/l	869	695	1043	Beckman DxC
	μg/ml	125	100	150	
	μmol/l	946	757	1135	Beckman AU400/600/640/2700/5400
	μg/ml	137	109	165	
	μmol/l	1069	855	1283	Siemens ADVIA 1200/1650/1800/2400
	μg/ml	154	123	185	
	μmol/l	1116	893	1339	Gravimetric
	μg/ml	161	129	193	
万古霉素	μmol/l	11.1	8.88	13.3	Roche Integra
	μg/ml	16.5	13.2	19.8	
	μmol/l	9.36	7.49	11.2	Ortho Vitros Microslide Systems
	μg/ml	13.9	11.1	16.7	
	μmol/l	10.9	8.72	13.1	Abbott Architect
	μg/ml	16.2	13.0	19.4	
	μmol/l	8.98	7.18	10.8	Siemens Centaur XP/XPT/Classic
	μg/ml	13.3	10.6	16.0	
	μmol/l	9.94	7.95	11.9	Siemens Dimension
	μg/ml	14.8	11.8	17.8	
	μmol/l	9.91	7.93	11.9	Roche Cobas 6000/8000
	μg/ml	14.7	11.8	17.6	
	μmol/l	9.89	7.91	11.9	Beckman DxC
	μg/ml	14.7	11.8	17.6	
	μmol/l	10.4	8.32	12.5	Thermo Scientific QMS Vancomycin
	μg/ml	15.5	12.4	18.6	
	μmol/l	10.8	8.64	13.0	Gravimetric
	μg/ml	16.0	12.8	19.3	
维生素 B12	pmol/l	710	568	852	Abbott Architect
	pg/ml	962	770	1154	
	pmol/l	852	639	1065	Siemens Centaur XP/XPT/Classic
	pg/ml	1154	866	1442	
	pmol/l	647	518	776	Siemens Immulite 2000
	pg/ml	877	702	1052	
	pmol/l	643	514	772	Beckman Dxl800
	pg/ml	871	696	1046	
	pmol/l	747	598	896	Roche Elecsys
	pg/ml	1012	810	1214	
	pmol/l	659	527	791	Beckman Access
	pg/ml	893	714	1072	
	pmol/l	866	693	1039	Tosoh Series
	pg/ml	1173	939	1407	
	pmol/l	632	506	758	Vitros ECI
	pg/ml	856	686	1026	



免疫质控品-水平3

货号：IA3111/IA3112 批号：1942EC 规格：12 x 5ml / 4 x 5ml 效期：2022-12-28

范围					
分析物	单位	靶值	低值	高值	方法学
维生素 B12	pmol/l	793	634	952	MP Biomedicals RIA
	pg/ml	1075	859	1291	
	pmol/l	711	569	853	Siemens Dimension
	pg/ml	963	771	1155	
	pmol/l	848	636	1060	Siemens Centaur CP
	pg/ml	1149	862	1436	