

凝血质控品

货号: CG5022

包装: 12 x 1 ml

批号: 357CG

效期: 2023-03

产品描述

本品适用于体外诊断中凝血检测系统的质控，用于监测凝血测定分析的正确度或重复度。朗道供应 3 个浓度水平（水平 1：CG5021；水平 2：CG5022；水平 3：CG5023）。

安全预防措施和警告

本产品仅用于体外诊断。禁止用口吸。按照实验室常规预防措施对试剂进行处理。

该质控品采用人源性血浆，对所有人源性材料均进行了 HIV（HIV1、HIV2）抗体、肝炎 B 表面抗原（HbsAg）和肝炎 C 病毒（HCV）抗体的测试，发现均呈阴性。所采用的方法均经 FDA 认证。

然而，既然没有一种方法能够完全保证其没有传染物质，因此该质控品和所有的病人样品均应当按照能够传播疾病的样品小心处理。

保存和稳定性

开瓶后，2~8℃ 保存。复溶后的血清如果保存在原始容器，并且无污染，活化部分凝血活酶时间（APTT）、凝血酶时间（TT）、凝血酶原时间（PT）、纤维蛋白原（FIB）、抗凝血酶Ⅲ（ATⅢ）可在 2~8℃ 保存 24 小时。C 蛋白、S 蛋白、血纤维蛋白溶酶原、因子Ⅱ、因子Ⅴ、因子Ⅶ、因子Ⅷ、因子Ⅸ、因子Ⅹ、因子Ⅺ、因子Ⅻ在 2~8℃ 保存可稳定 8 小时。每次使用只吸取所需用量，剩余的样品不可返回原瓶。

未开瓶，2~8 °C 可保存至效期末。

使用说明

该凝血质控品为冻干粉。

1. 小心打开瓶盖，往瓶中准确加入 1 ml 蒸馏水，在 15~25 °C 的条件下复溶，盖上瓶盖，放置 30 分钟。轻轻翻转瓶子，确保内容物完全溶解，不要振荡，避免气泡的产生。
2. 参照各个分析仪的质控说明。
3. 冷藏未使用的材料，再次使用前，确保样品充分混合均匀。

需要自备的材料

移液管

赋值

每一批质控品都要送到大量的外部实验室，然后将结果与朗道实验室内得到的结果用相同的统计分析进行赋值。在实验室建立起自己方法学的靶值和 SD 范围前，质控范围值用来协助实验室进行质量控制。

注：详细赋值信息请以原版英文说明书为准，原版说明书请在英国朗道公司官网 www.randox.com 进行下载。



凝血质控品-水平 2

货号：CG5022		批号：357CG		规格：12 x 1 ml		效期：2023-03		
		范围			低值=靶值-2SD		高值=靶值+2SD	
分析物	单位	靶值	低值	高值	方法学			
活化部分凝血活酶时间（APTT）	seconds	53.3	42.6	64.0	Tcoag TriniCLOT aPTT S			
	seconds	51.0	40.8	61.2	Tcoag TriniCLOT aPTT HS			
	seconds	45.4	36.3	54.5	Siemens/Dade Actin			
	seconds	44.0	35.2	52.8	Siemens/Dade Actin FS			
	seconds	40.9	32.7	49.1	Siemens/Dade Actin FSL			
	seconds	77.8	62.2	93.4	Siemens/Dade Actin SL			
	seconds	76.0	60.8	91.2	Siemens/Dade Pathromtin SL			
	seconds	42.6	34.1	51.1	HemosIL APTT-SP Liquid			
	seconds	59.3	47.4	71.2	HemosIL Synthasil			
	seconds	66.1	52.9	79.3	Pacific Hemostasis APTT-XL			
	seconds	52.6	42.1	63.1	Stago aPTT			
	seconds	47.4	37.9	56.9	Stago Cephascreen			
	seconds	55.9	44.7	67.1	Stago CK-Prest			
	seconds	54.2	43.4	65.0	Stago PTT Automate			
	seconds	53.2	42.6	63.8	Teco TEClot APTT			
	seconds	47.0	37.6	56.4	Human Hemostat APTT			
	seconds	48.4	38.7	58.1	Bio-Ksel System APTTs			
	seconds	59.5	47.6	71.4	Diagon Dia-PTT			
	seconds	46.6	37.3	55.9	Weiner Lab APTT			
	seconds	52.1	41.7	62.5	Biolabo Bio-CK APTT			
	seconds	51.1	40.9	61.3	Spinreact APTT			
	seconds	58.0	46.4	69.6	Shanghai Changdao Biotech APTT			
	seconds	47.8	38.2	57.4	Technology-Standard aPTT			
	seconds	44.6	35.7	53.5	Biomedica QuikCoag aPTT			
	seconds	47.4	37.9	56.9	Siemens/Dade Actin Cephaloplastin			
	seconds	46.2	37.0	55.4	Labitec aPTT			
	seconds	46.8	37.4	56.2	Dutch Diagnostics aPTT			
	seconds	53.8	43.0	64.6	Sclavo APTT-S			
	seconds	59.7	47.8	71.6	HemoDiagnostics aPTT			
	seconds	47.8	38.2	57.4	Fortress Diagnostics aPTT			
	seconds	57.8	46.2	69.4	Tokra Medikal APTT			
	seconds	37.1	29.7	44.5	Coagpia APTT-N			
	seconds	74.9	59.9	89.9	Cobas aPTT MedS			
	seconds	49.7	39.8	59.6	Diagon Dia-PTT Liquid			
	seconds	43.5	34.8	52.2	PRECIL			
	seconds	48.9	39.1	58.7	Erba Actime APTT			
	seconds	49.2	39.4	59.0	MTI Diagnostics APTT			
	seconds	46.8	37.4	56.2	Rayto aPTT Reagent			
	seconds	73.3	58.6	88.0	Roche Cobas APTT Screen			
	seconds	67.9	54.3	81.5	Roche Cobas APTT			
活化部分凝血活酶时间	ratio	1.73	1.38	2.08	Tcoag TriniCLOT aPTT S			
	ratio	1.53	1.22	1.84	Siemens/Dade Actin FSL			



凝血质控品-水平 2

货号：CG5022		批号：357CG		规格：12 x 1 ml		效期：2023-03	
		范围		低值=靶值-2SD		高值=靶值+2SD	
分析物	单位	靶值	低值	高值	方法学		
活化部分凝血活酶时间（APTT）比	ratio	1.59	1.27	1.91	Stago Cephascreen		
	ratio	1.66	1.33	1.99	Stago PTT Automate		
	ratio	1.70	1.36	2.04	Stago aPTT		
	ratio	2.41	1.93	2.89	Siemens/Dade Pathromtin SL		
	ratio	1.45	1.16	1.74	HemosIL APTT-SP Liquid		
	ratio	1.95	1.56	2.34	HemosIL Synthasil		
	ratio	1.81	1.45	2.17	Stago CK-Prest		
	ratio	1.66	1.33	1.99	Teco TEClot APTT		
	ratio	1.70	1.36	2.04	Human Hemostat APTT		
	ratio	1.40	1.12	1.68	Bio-Ksel System APTTs		
	ratio	1.70	1.36	2.04	Siemens/Dade Actin FS		
	ratio	1.86	1.49	2.23	Diagon Dia-PTT		
	ratio	1.79	1.43	2.15	Spinreact APTT		
	ratio	1.59	1.27	1.91	Media IVD aPTT		
	ratio	1.51	1.21	1.81	Labitec aPTT		
	ratio	1.91	1.53	2.29	Sclavo APTT-S		
	ratio	1.74	1.39	2.09	Diagon Dia-PTT Liquid		
	ratio	1.62	1.30	1.94	Erba Actime APTT		
	ratio	1.62	1.30	1.94	MTI Diagnostics APTT		
	抗凝血酶Ⅲ（AT Ⅲ）活性	%	66.1	52.9	79.3	Siemens/Dade Berichrom ATIII	
%		55.0	44.0	66.0	Helena Chrom-Z		
%		61.9	49.5	74.3	HemosIL Liquid Antithrombin		
%		66.7	53.4	80.0	Roche Chronogenic Thrombin		
%		68.4	54.7	82.1	Hemosil Lyophilised Antithrombin III		
%		69.4	55.5	83.3	STACHrom ATIII		
%		64.7	51.8	77.6	Siemens/Dade Antithrombin III		
%		64.4	51.5	77.3	Siemens Innovance Antithrombin III		
%		65.1	52.1	78.1	Sclavo ATIII		
%		66.2	53.0	79.4	Roche Antithrombin		
%		60.9	48.7	73.1	HemoDiagnostics Antithrombin III (Fxa)		
Ⅱ 因子活性	%	76.8	61.4	92.2	Siemens/Dade-Behring Innovin		
	%	75.9	60.7	91.1	Siemens/Dade-Behring Thromborel S		
	%	74.3	59.4	89.2	HemosIL PT-Fibrinogen HS Plus		
	%	86.5	69.2	104	HemosIL RecombiPlasTin 2G		
	%	85.4	68.3	102	HemosIL Factor II deficient plasma		
	%	69.6	55.7	83.5	Stago STA-NeoPTimal		
Ⅸ 因子活性	%	61.5	49.2	73.8	Stago PTT Automate		
	%	58.3	46.6	70.0	Siemens/Dade Actin FS		
	%	66.6	53.3	79.9	Siemens/Dade Pathromtin SL		
	%	59.1	47.3	70.9	HemosIL Synthasil		
	%	69.0	55.2	82.8	Stago CK-Prest		
	%	60.6	48.5	72.7	Siemens/Dade Actin FSL		
	%	68.3	54.6	82.0	Stago Deficient / ImmunoDef IX		
	%	58.0	46.4	69.6	HemosIL Factor IX deficient plasma		
Ⅴ 因子活性	%	58.0	46.4	69.6	Siemens/Dade-Behring Thromborel S		
	%	50.8	40.6	61.0	Siemens/Dade-Behring Innovin		

凝血质控品-水平 2

货号: CG5022 批号: 357CG 规格: 12 x 1 ml 效期: 2023-03					
		范围		低值=靶值-2SD	高值=靶值+2SD
分析物	单位	靶值	低值	高值	方法学
V 因子活性	%	56.0	44.8	67.2	HemosIL PT-Fibrinogen HS Plus
	%	53.6	42.9	64.3	HemosIL RecombiPlasTin 2G
	%	53.2	42.6	63.8	HemosIL Factor V deficient plasma
VII 因子活性	%	132	106	158	Siemens/Dade-Behring Thromborel S
	%	168	134	202	Siemens/Dade-Behring Innovin
	%	218	174	262	HemosIL PT-Fibrinogen HS Plus
	%	198	158	238	HemosIL RecombiPlasTin 2G
	%	193	154	232	HemosIL Factor VII deficient plasma
	%	188	150	226	Stago Deficient Factor VII
	%	153	122	184	Stago STA-NeoPTimal
VII 因子活性	%	41.8	33.4	50.2	Stago CK-Prest
	%	35.2	28.2	42.2	Stago PTT Automate
	%	40.5	32.4	48.6	Siemens/Dade Actin FS
	%	42.2	33.8	50.6	Siemens/Dade Pathromtin SL
	%	40.7	32.6	48.8	Siemens/Dade Actin FSL
	%	39.3	31.4	47.2	HemosIL Synthasil
	%	37.9	30.3	45.5	HemosIL APTT-SP Liquid
	%	36.0	28.8	43.2	Stago aPTT
	%	42.8	34.2	51.4	Stago Deficient / ImmunoDef VIII
	%	39.3	31.4	47.2	HemosIL Factor VIII deficient plasma
X 因子活性	%	92.5	74.0	111	Siemens/Dade-Behring Thromborel S
	%	98.7	79.0	118	Siemens/Dade-Behring Innovin
	%	88.8	71.0	107	HemosIL PT-Fibrinogen HS Plus
	%	82.3	65.8	98.8	Stago Neoplastine CI Plus
	%	86.8	69.4	104	HemosIL RecombiPlasTin 2G
	%	83.5	66.8	100	HemosIL Factor X deficient plasma
	%	82.0	65.6	98.4	Stago STA-NeoPTimal
XI 因子活性	%	36.7	29.4	44.0	HemosIL Synthasil
	%	29.8	23.8	35.8	Siemens/Dade Actin FSL
	%	33.9	27.1	40.7	HemosIL Factor XI Deficient Plasma
XII 因子活性	%	76.0	60.8	91.2	Siemens/Dade Actin FS
	%	67.9	54.3	81.5	Siemens/Dade Pathromtin SL
	%	66.7	53.4	80.0	HemosIL Synthasil
	%	64.1	51.3	76.9	Siemens/Dade Actin FSL
	%	68.9	55.1	82.7	HemosIL APTT-SP Liquid
	%	64.7	51.8	77.6	HemosIL Factor XII deficient plasma
纤维蛋白原	g/L	1.67	1.34	2.00	HemosIL PT-Fibrinogen HS Plus
	mg/dL	167	134	200	
	g/L	1.81	1.45	2.17	Helena Fibrinogen
	mg/dL	181	145	217	
	g/L	1.85	1.48	2.22	Stago Fibrinogen / Liquid-Fib
	mg/dL	185	148	222	
	g/L	1.58	1.26	1.90	Siemens/Dade Multifibren U
	mg/dL	158	126	190	
	g/L	1.75	1.40	2.10	HemosIL Fibrinogen C
	mg/dL	175	140	210	



凝血质控品-水平 2

货号：CG5022		批号：357CG		规格：12 x 1 ml		效期：2023-03
		范围		低值=靶值-2SD	高值=靶值+2SD	
分析物	单位	靶值	低值	高值	方法学	
纤维蛋白原	g/L	1.61	1.29	1.93	Dade Thrombin	
	mg/dL	161	129	193		
	g/L	2.20	1.76	2.64	Technoclone Fibrinogen	
	mg/dL	220	176	264		
	g/L	1.82	1.46	2.18	Tcoag TriniCLOT Fibrinogen	
	mg/dL	182	146	218		
	g/L	1.80	1.44	2.16	Biolabo Bio-Fibri	
	mg/dL	180	144	216		
	g/L	1.67	1.34	2.00	Teco Teclot Fibrinogen	
	mg/dL	167	134	200		
	g/L	1.72	1.38	2.06	HemosIL QFA (bovine thrombin)	
	mg/dL	172	138	206		
	g/L	1.93	1.54	2.32	Human Fibrinogen	
	mg/dL	193	154	232		
	g/L	1.84	1.47	2.21	Stago STA Fibri-Prest Automate	
	mg/dL	184	147	221		
	g/L	1.14	0.912	1.37	Bio-Ksel System PT	
	mg/dL	114	91.2	137		
	g/L	1.83	1.46	2.20	Bio-Ksel Fibrinogen	
	mg/dL	183	146	220		
	g/L	1.63	1.30	1.96	Diagon Dia-F	
	mg/dL	163	130	196		
	g/L	1.28	1.02	1.54	Dade Innovin Derived Fibrinogen	
	mg/dL	128	102	154		
	g/L	1.27	1.02	1.52	Siemens/Dade Thromboplastin Derived Fibrinogen	
	mg/dL	127	102	152		
	g/L	1.81	1.45	2.17	Wiener Lab Fibrinogen	
	mg/dL	181	145	217		
	g/L	1.78	1.42	2.14	Spinreact Fibrinogen	
	mg/dL	178	142	214		
	g/L	1.19	0.952	1.43	HemosIL RecombiPlasTin 2G	
	mg/dL	119	95.2	143		
	g/L	1.78	1.42	2.14	Sclavo Fibrinogen	
	mg/dL	178	142	214		
	g/L	1.82	1.46	2.18	Labitec Fibrinogen	
	mg/dL	182	146	218		
	g/L	1.79	1.43	2.15	HemoDiagnostics Fibrinogen	
	mg/dL	179	143	215		
	g/L	1.82	1.46	2.18	Helena Clauss Fibrinogen 100	
	mg/dL	182	146	218		
	g/L	2.18	1.74	2.62	Tokra Medikal Fibrinogen	
	mg/dL	218	174	262		
	g/L	1.81	1.45	2.17	Coagpia Fibrinogen	
	mg/dL	181	145	217		
	g/L	1.72	1.38	2.06	Cobas Fibrinogen	
	mg/dL	172	138	206		



凝血质控品-水平 2

货号：CG5022		批号：357CG		规格：12 x 1 ml		效期：2023-03
		范围		低值=靶值-2SD	高值=靶值+2SD	
分析物	单位	靶值	低值	高值	方法学	
纤维蛋白原	g/L	1.64	1.31	1.97	PRECIL	
	mg/dL	164	131	197		
	g/L	1.63	1.30	1.96	Erba Thrombin Reagent	
	mg/dL	163	130	196		
	g/L	1.62	1.30	1.94	Yumizen G Fibrinogen	
	mg/dL	162	130	194		
	g/L	1.72	1.38	2.06	Roche Cobas t511/t711 Fibrinogen	
	mg/dL	172	138	206		
	g/L	1.73	1.38	2.08	Roche Cobas t411 Fibrinogen	
	mg/dL	173	138	208		
血浆酶原活性	%	69.2	55.4	83.0	HemosIL Plasminogen	
	%	78.6	62.9	94.3	Siemens/Dade Berichrom Plasminogen	
C 蛋白	%	69.3	55.4	83.2	Siemens/Dade Berichrom Protein C	
	%	70.3	56.2	84.4	Stago StaClot	
	%	60.4	48.3	72.5	Biomerieux VIDAS Protein C	
	%	65.0	52.0	78.0	IL HemosIL Protein C	
	%	73.6	58.9	88.3	Siemens	
	%	69.8	55.8	83.8	STACHrom Protein C	
S 蛋白	%	66.2	46.3	86.1	STA Liatest	
	%	55.4	38.8	72.0	Stago StaClot	
	%	73.7	51.6	95.8	IL HemosIL ProS	
	%	69.2	48.4	90.0	IL HemosIL Free Protein S	
	%	50.8	35.6	66.0	Dade Protein S Ac	
	%	63.0	44.1	81.9	Siemens Innovance Free PS Ag	
凝血酶原时间（PT）	%	33.8	23.7	43.9	IL HemosIL Protein S Activity	
	Seconds	15.5	12.4	18.6	Siemens/Dade-Behring Innovin	
	Seconds	14.7	11.8	17.6	Siemens/Dade Thromborel IS	
	Seconds	13.9	11.1	16.7	Siemens/Dade-Behring Thromborel S	
	Seconds	17.3	13.8	20.8	HemosIL PT-Fibrinogen HS Plus	
	Seconds	17.9	14.3	21.5	Pacific Thromboplastin DS	
	Seconds	17.5	14.0	21.0	Stago Neoplastine CI	
	Seconds	17.4	13.9	20.9	Stago Neoplastine CI Plus	
	Seconds	15.8	12.6	19.0	Teco TEClot PT-S	
	Seconds	17.2	13.8	20.6	Biolabo Bio-TP	
	Seconds	14.0	11.2	16.8	Stago Neoplastin R	
	Seconds	11.6	9.28	13.9	Bio-Ksel System PT	
	Seconds	15.1	12.1	18.1	Human Thromboplastin	
	Seconds	15.7	12.6	18.8	Diagon PT	
	Seconds	25.9	20.7	31.1	Weiner Lab Soluplastin	
	Seconds	17.1	13.7	20.5	Spinreact PT	
	Seconds	14.2	11.4	17.0	Shanghai Changdao Biotech PT	
	Seconds	15.1	12.1	18.1	Technology-Standard PT	
	Seconds	21.1	16.9	25.3	TCoag TriniCLOT PT Excel S	
	Seconds	16.5	13.2	19.8	Media IVD PT	
	Seconds	14.3	11.4	17.2	Biodevice PT	
	Seconds	16.4	13.1	19.7	Biomedica QuikCoag PT-HS	



凝血质控品-水平 2

货号：CG5022		批号：357CG		规格：12 x 1 ml		效期：2023-03	
		范围		低值=靶值-2SD		高值=靶值+2SD	
分析物	单位	靶值	低值	高值	方法学		
凝血酶原时间（PT）	Seconds	11.7	9.36	14.0	HemosIL RecombiPlasTin 2G		
	Seconds	16.3	13.0	19.6	Helena Thromboplastin LI		
	Seconds	16.7	13.4	20.0	Helena Thromboplastin		
	Seconds	14.5	11.6	17.4	Labitec PT		
	Seconds	13.0	10.4	15.6	Dutch Diagnostics PT		
	Seconds	15.2	12.2	18.2	Sclavo PT		
	Seconds	17.4	13.9	20.9	Haemo Diagnostics Thromboplastin		
	Seconds	16.5	13.2	19.8	Cypress Diagnostics PT		
	Seconds	14.4	11.5	17.3	Fortress Diagnostics PT with Calcium		
	Seconds	15.4	12.3	18.5	Tokra Medikal PT		
	Seconds	14.9	11.9	17.9	Coagpia PT-N		
	Seconds	11.4	9.12	13.7	Bio-Ksel PT Plus		
	Seconds	15.6	12.5	18.7	Beijing Succeder PT		
	Seconds	11.3	9.04	13.6	Diagon PT R		
	Seconds	15.0	12.0	18.0	Cobas PT Screen		
	Seconds	15.2	12.2	18.2	PRECIL		
	Seconds	20.3	16.2	24.4	Stago STA-NeoPTimal		
	Seconds	19.4	15.5	23.3	MTI Diagnostics Prothrombin Time		
	Seconds	15.8	12.6	19.0	Yumizen G PT		
	Seconds	15.6	12.5	18.7	Erba Protime PT		
Seconds	12.1	9.68	14.5	HemosIL ReadiPlasTin			
Seconds	16.3	13.0	19.6	Roche Cobas PT Rec			
凝血酶原时间（PT） 活性	%	44.1	35.3	52.9	Siemens/Dade-Behring Innovin		
	%	72.4	57.9	86.9	Siemens/Dade-Behring Thromborel S		
	%	62.7	50.2	75.2	Stago Neoplastine CI		
	%	64.7	51.8	77.6	Stago Neoplastine CI Plus		
	%	72.6	58.1	87.1	HemosIL PT-Fibrinogen HS Plus		
	%	72.8	58.2	87.4	Teco TEClot PT-S		
	%	52.6	42.1	63.1	Biolabo Bio-TP		
	%	88.9	71.1	107	Stago Neoplastin R		
	%	69.2	55.4	83.0	Human Thromboplastin		
	%	69.2	55.4	83.0	Diagon PT		
	%	97.0	77.6	116	Technology-Standard PT		
	%	40.4	32.3	48.5	TCoag TriniCLOT PT Excel S		
	%	78.2	62.6	93.8	Biodevice PT		
	%	93.9	75.1	113	HemosIL RecombiPlasTin 2G		
	%	65.0	52.0	78.0	Helena Thromboplastin		
	%	69.7	55.8	83.6	Labitec PT		
	%	78.8	63.0	94.6	Sclavo PT		
	%	76.7	61.4	92.0	Tokra Medikal PT		
	%	96.8	77.4	116	Bio-Ksel PT Plus		
	%	63.2	50.6	75.8	Cobas PT Screen		
	%	70.2	56.2	84.2	PRECIL		
	%	54.0	43.2	64.8	Stago STA-NeoPTimal		
	%	72.4	57.9	86.9	Erba Protime PT		
	%	45.7	36.6	54.8	MTI Diagnostics Prothrombin Time		



凝血质控品-水平 2

货号：CG5022		批号：357CG		规格：12 x 1 ml		效期：2023-03	
		范围			低值=靶值-2SD	高值=靶值+2SD	
分析物	单位	靶值	低值	高值	方法学		
凝血酶原时间（PT） 活性	%	93.9	75.1	113	HemosIL ReadiPlasTin		
	%	37.6	30.1	45.1	Roche Cobas PT Rec		
凝血酶原时间 （PT）INR	INR	1.54	1.23	1.85	Siemens/Dade-Behring Innovin		
	INR	1.24	0.992	1.49	Siemens/Dade Thromborel IS		
	INR	1.21	0.968	1.45	Siemens/Dade-Behring Thromborel S		
	INR	1.27	1.02	1.52	HemosIL PT-Fibrinogen		
	INR	1.23	0.984	1.48	HemosIL PT-Fibrinogen HS Plus		
	INR	1.43	1.14	1.72	Pacific Thromboplastin DS		
	INR	1.41	1.13	1.69	Stago Neoplastine CI		
	INR	1.37	1.10	1.64	Stago Neoplastine CI Plus		
	INR	1.19	0.952	1.43	Human Thromboplastin		
	INR	1.26	1.01	1.51	Teco TEClot PT-S		
	INR	1.73	1.38	2.08	Biolabo Bio-TP		
	INR	1.05	0.840	1.26	Stago Neoplastin R		
	INR	1.04	0.832	1.25	Bio-Ksel System PT		
	INR	1.22	0.976	1.46	Diagon PT		
	INR	2.45	1.96	2.94	Weiner Lab Soluplastin		
	INR	1.44	1.15	1.73	Futura System Thromboplastin S		
	INR	1.50	1.20	1.80	Spinreact PT		
	INR	1.24	0.992	1.49	Technology-Standard PT		
	INR	1.74	1.39	2.09	TCoag TriniCLOT PT Excel S		
	INR	1.20	0.960	1.44	Media IVD PT		
	INR	1.16	0.928	1.39	Biodevice PT		
	INR	1.31	1.05	1.57	Biomedica QuikCoag PT-HS		
	INR	1.04	0.832	1.25	HemosIL RecombiPlasTin 2G		
	INR	1.15	0.920	1.38	Technoclon Technoplastin HIS		
	INR	1.40	1.12	1.68	Helena Thromboplastin		
	INR	1.26	1.01	1.51	Labitec PT		
	INR	1.21	0.968	1.45	Sclavo PT		
	INR	1.40	1.12	1.68	Haemo Diagnostics Thromboplastin		
	INR	1.24	0.992	1.49	Fortress Diagnostics PT with Calcium		
	INR	1.17	0.936	1.40	Technoclon Technoclot PT Plus		
	INR	1.14	0.912	1.37	Tokra Medikal PT		
	INR	1.24	0.992	1.49	Coagpia PT-N		
	INR	1.03	0.824	1.24	Bio-Ksel PT Plus		
	INR	1.34	1.07	1.61	Beijing Succeeder PT		
	INR	1.09	0.872	1.31	Diagon PT R		
	INR	1.42	1.14	1.70	Cobas PT Screen		
	INR	1.27	1.02	1.52	PRECIL		
	INR	1.55	1.24	1.86	Stago STA-NeoPTimal		
	INR	1.21	0.968	1.45	Erba Protime PT		
	INR	1.69	1.35	2.03	MTI Diagnostics Prothrombin Time		
	INR	1.26	1.01	1.51	Yumizen G PT Reco		
	INR	1.08	0.864	1.30	Rayto Prothrombin Time Reagent		
	INR	1.05	0.840	1.26	HemosIL ReadiPlasTin		
	INR	1.76	1.41	2.11	Roche Cobas PT Rec		



凝血质控品-水平 2

货号：CG5022		批号：357CG		规格：12 x 1 ml		效期：2023-03	
		范围		低值=靶值-2SD		高值=靶值+2SD	
分析物	单位	靶值	低值	高值	方法学		
凝血酶原时间（PT） 比率	ratio	1.54	1.23	1.85	Siemens/Dade-Behring Innovin		
	ratio	1.20	0.960	1.44	Siemens/Dade-Behring Thromborel S		
	ratio	1.45	1.16	1.74	Stago Neoplastine CI Plus		
	ratio	1.18	0.944	1.42	Diagon PT		
	ratio	1.15	0.920	1.38	HemosIL PT-Fibrinogen HS Plus		
	ratio	1.54	1.23	1.85	TCoag TriniCLOT PT Excel S		
	ratio	1.04	0.832	1.25	HemosIL RecombiPlasTin 2G		
	ratio	1.25	1.00	1.50	Helena Thromboplastin		
	ratio	1.15	0.920	1.38	Sclavo PT		
	ratio	1.54	1.23	1.85	Stago STA-NeoPTimal		
凝血酶时间（TT）	Seconds	29.6	23.7	35.5	Dade Thrombin		
	Seconds	23.0	18.4	27.6	Siemens/Dade Thromboclotin		
	Seconds	28.1	22.5	33.7	Stago Thrombin		
	Seconds	30.9	24.7	37.1	Sysmex Thrombin Time		
	Seconds	26.7	21.4	32.0	Diagon, Dia-TT		
	Seconds	20.8	16.6	25.0	Helena Thrombin Clotting Time rgt		
	Seconds	22.8	18.2	27.4	HemosIL Thrombin Time 5ml		
	Seconds	27.1	21.7	32.5	HemosIL Thrombin Time 8ml		
	Seconds	33.6	26.9	40.3	Siemens BC Thrombin		
	Seconds	19.3	15.4	23.2	PRECIL		